

**Installation Guide** Part #: 70-0286 (See DSU-H Instructions for Use)


## DSU-H Shower

**AVOID contact between filter and cleaning or sanitizing agents.<sup>(1)</sup>**

**DO NOT remove blister pack from filter until indicated to protect from contamination.**

Questions or issues?  
Let us help!


Call:  
**(201) 343-5202, Ext. 1**

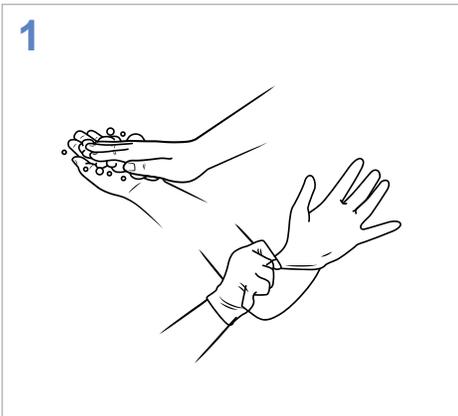

**1** READ the entire instruction guide and precautions before starting.


WASH hands before starting installation.

WEAR gloves before handling filter (do not remove blister pack).

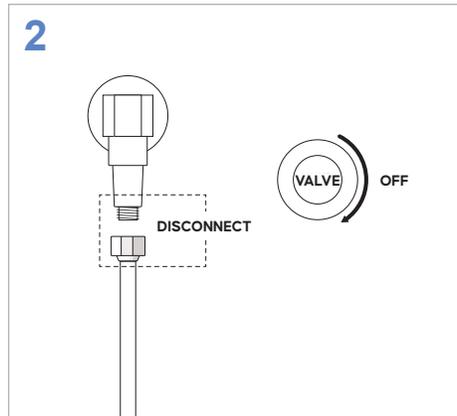


**2** TURN OFF the shower valve.


DISCONNECT and REMOVE the existing shower head and hose from the water supply pipe.

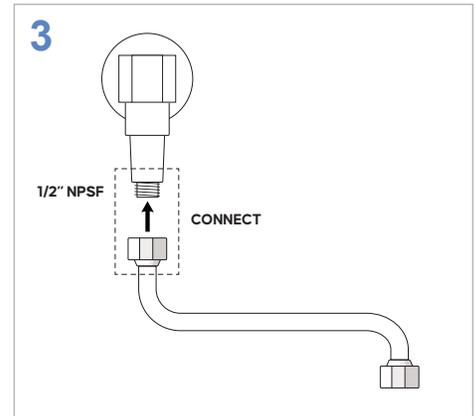


**3** CONNECT a new flex line by the 1/2" NPSF fitting to the water supply pipe.

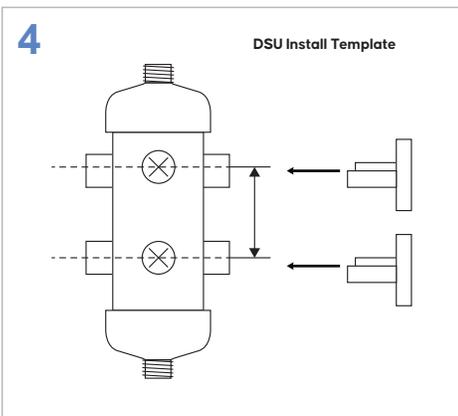

**4** ATTACH the DSU Template (provided in installation kit) vertically against the wall in a desired position to ensure that both flex lines are not stretched.

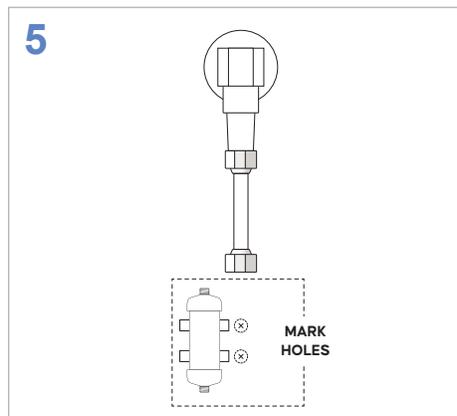

**5** MARK the hole locations of the C-Clips at their locations.

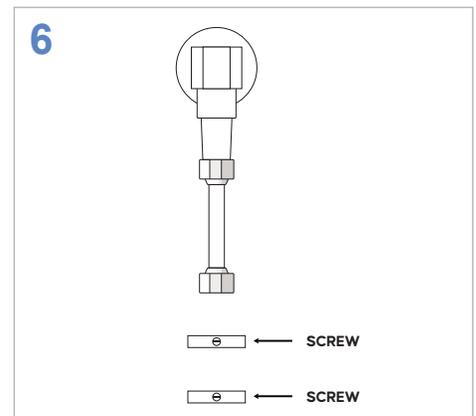

**6** MOUNT the C-Clips with 2 screws (1 each).

(1) Medical grade plastics are more susceptible to cracking due to exposure to organic solvents. Alkyl dimethyl benzyl ammonium chloride (ADBAC), glycols, alcohols, ketones, silicones, and acetones are some examples of the leading organic solvents found in cleaning and sanitizing agents. Use of cleaning/sanitizing agents on the filter housings is not required and should be avoided.

**Installation Guide** Part #: 70-0286 (See DSU-H Instructions for Use)


## DSU-H Shower (continued)

**AVOID contact between filter and cleaning or sanitizing agents.**

**DO NOT remove blister pack from filter until indicated to protect from contamination.**

Questions or issues?  
Let us help!


Call:  
**(201) 343-5202, Ext. 1**

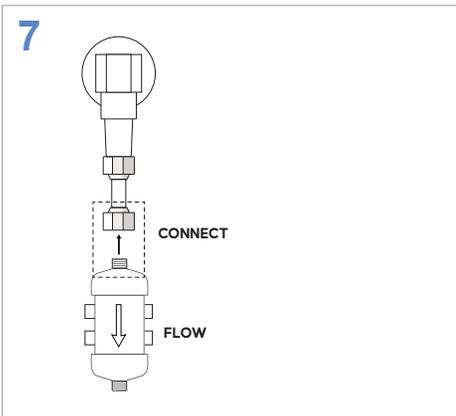

**DO NOT REMOVE** the blister pack. While wearing gloves, **HOLD** the filter by the transparent portion of the blister pack and tear away the white backing as shown in Diagram 1.


**REMOVE** blister pack and **CLIP** the filter to C-Clips so the **FLOW** direction is towards the flow of water, pointing away from the water supply pipe.

**CONNECT** the water supply pipe flex line to the inlet of the filter.

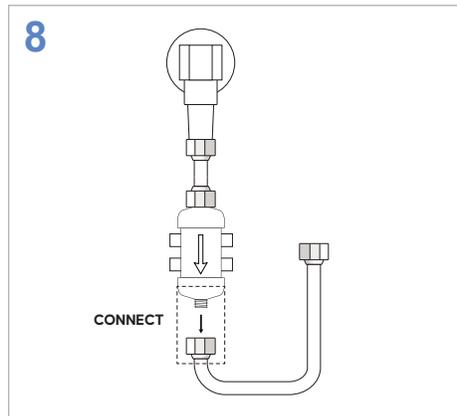

**CONNECT** the shower hose to the outlet of the filter.

Refer to either Option 1 or Option 2 for connection to showerhead in next step.

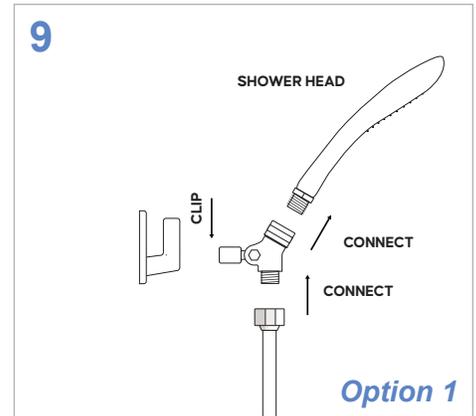

Option 1:


**CONNECT** the female connector on the shower hose to the Swivel Connector.

**CONNECT** the male connector on the swivel connector to the shower head.

**CLIP** the attachment on the swivel connector onto the existing shower hook.

**TURN ON** the water supply and allow water to run for 3-5 minutes.

**CHECK** for leaks and tighten any fittings if necessary.

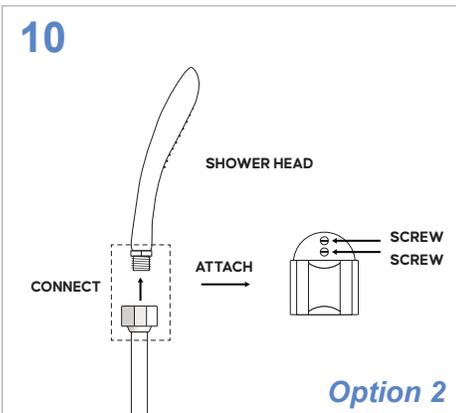

Option 2:

**CONNECT** the other end of the shower hose to the hand shower head.

**MOUNT** the shower holder by its screws at desired positions for holding the hand shower head.

**TURN ON** the water supply and allow water to run for 3-5 minutes to flush the filter and purge the trapped air.

**CHECK** for leaks and tighten any fittings if necessary.



Diagram 1

NEPHROS.COM

**MANUFACTURER:**

Nephros, Inc.  
380 Lackawanna Place, South Orange, NJ 07079 USA  
(201) 343-5202

**ASSEMBLED AT:**

MEDICA S.p.A.  
Via Degli Artigiani, 7  
41036 Medolla (MO), Italy