


**Installation Guide** Part #: 70-0280 / 70-0281 (See S100 Instructions for Use)

## S100 Point of Use Sink




**1**

WASH hands before starting installation.  
WEAR gloves before handling filter.  
Hand sanitizers used in healthcare can affect performance of product material.<sup>(1)</sup>

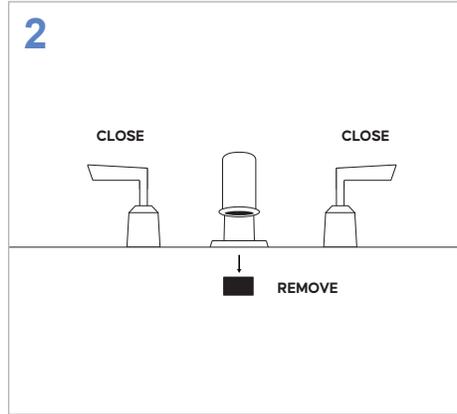


**2**

CLOSE the water valves.  
REMOVE the aerator from the faucet.  
If a Point of Use filter is currently installed, press and hold the tab of the aerator adapter to release the filter.  
To dispose of an old Point of Use filter, do so according to the facility policy.

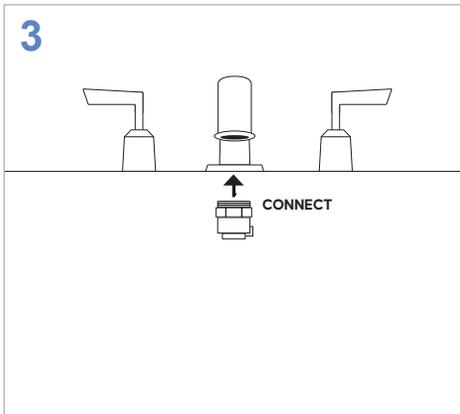

**3**

If an aerator adapter needs to be installed, CONNECT the aerator adapter into the faucet opening. The aerator adapter thread size should match the faucet thread (e.g. 55/64" or 15/16").  
Keep in mind that once installed to the adapter, the bottom of the filter must be at least 1 inch above the top rim of the sink. A filter must not be installed if its discharge is below this height.  
OPEN the faucet valves and be sure there is no leaking at the aerator adapter connection site. Tighten adapter if needed to eliminate leaking of water.  
CLOSE the valves after checking.

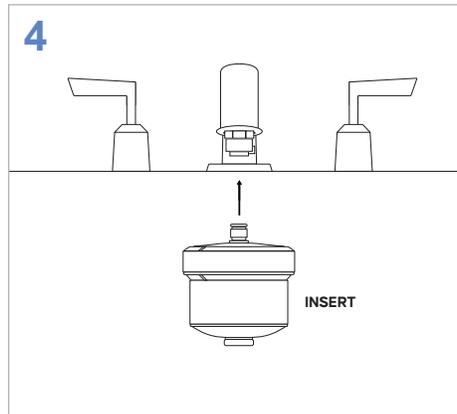

**4**

INSERT the S100 Point of Use filter into the aerator adapter.  
PUSH the filter in until the latch clicks and locks the filter fully in place.  
OPEN the faucet valves slowly to be sure the filter is fully inserted into the aerator adapter (no leaking). When certain, use faucet valves as normal.

Questions or issues?  
Let us help!

Call:  
**(201) 343-5202, Ext. 1**

**AVOID contact between the filter and the chemical alkyl dimethyl benzyl ammonium chloride (ADBAC).<sup>(1)</sup>**

If external cleaning of the S100 is needed, use a 1% bleach cleaner such as Clorox® Healthcare Bleach Germicidal, or Dispatch® Hospital Cleaner Disinfectant Towels with Bleach.

All S100 filters contain a bacteriostatic additive incorporated into the housing material in order to guard against surface touch contamination. In cases where the external housing of the filters might be subject to heavy surface contamination (e.g. rinsing of used surgical instruments), it is recommended to wipe down the exterior surface of the filters with 1% bleach. Do not remove the filters or immerse them in cleaning solution.

<sup>(1)</sup> The chemical ADBAC can cause structural damage to filter.

NEPHROS.COM

**MANUFACTURER:**  
Nephros, Inc.  
380 Lackawanna Place, South Orange, NJ 07079 USA  
(201) 343-5202

**ASSEMBLED AT:**  
MEDICA S.p.A.  
Via Degli Artigiani, 7  
41036 Medolla (MO), Italy

50-3213 REV03